



STERILIZATION QUICK GUIDE

ENSURE PROPER USE OF YOUR STERILIZER. FOLLOW THESE STEPS FOR STERILITY ASSURANCE, **EVERY TIME.**

STEAM STERILIZERS

Steam sterilizers are recommended as the sterilization method of choice by the CDC



GRAVITY DISPLACEMENT STEAM STERILIZER

Manufacturer Models Include: TUTTNAUER™: Manual M/MK series (2340M, 2540M, 2540MK, 3870M) and Valueclave™ 1730 Fully Automated: EZ9™, EZ10™, EZ10k™, 3870EA

DYNAMIC AIR REMOVAL FOR STEAM FLUSH PRESSURE PULSE (SFPP)

Manufacturer Models Include: SCICAN: Statim 2000 / 5000 / 2000 G4 / 5000 G4. MIDMARK: M3 / M9 / M11. TUTTNAUER - EZ Plus Series (EZ9 Plus, EZ11 Plus)

1



Sure-Check® Pouches
For use with **EVERY LOAD**
(Internal/external multi-parameter chemical indicators)

2



STEAMPlus™ Type 5* Integrator
For use with **EVERY LOAD** or at least **EVERY DAY** for added sterility assurance

3



Mail-In or In-Office ConForm® Biological Monitors
Spore test **AT LEAST WEEKLY**

DYNAMIC AIR REMOVAL PRE-VACUUM (CLASS B)

Pre-vacuum (Class B) – Requires daily air removal test (Bowie-Dick) to verify vacuum pump is working properly.

Manufacturer Models Include: SCICAN: Bravo, W & H: Lisa, TUTTNAUER: Elara 11



Run Warm-up Cycle
with an empty chamber at the start of **EVERY DAY**

2



AirView™ II Bowie-Dick Test
Use in empty chamber after warm-up cycle **EVERY DAY**

3



Sure-Check® Pouches
For use with **EVERY LOAD**
(Internal/external multi-parameter chemical indicators)

4




STEAMPlus™ Type 5* Integrator
For use with **EVERY LOAD** or at least **EVERY DAY** for added sterility assurance

5



Mail-In or In-Office ConForm® Biological Monitors
Spore test **AT LEAST WEEKLY**

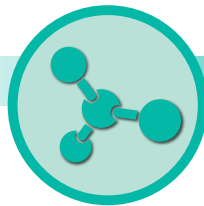

CHEMICAL VAPOR STERILIZERS

Chemical Vapor Sterilizers use formaldehyde solution instead of distilled water

Chemical-Vapor sterilizers look like, but do not act like steam sterilizers and are found primarily on the West Coast. Chemical-Vapor sterilizers are pre-set for 270°F for 20 minutes exposure. No dry time is needed. Maximum load is 3.2lbs.

1



Sure-Check® Chemical Indicator Strips
For use in **EVERY POUCH**

2



In-Office SporView® Culture Set
Spore test **AT LEAST WEEKLY**

-OR-



Mail-In ConForm® Biological Indicators
Spore test **AT LEAST WEEKLY**

DRY HEAT STERILIZERS

Used primarily in Ortho, Perio and Oral Surgery Practices

Dry Heat sterilizers are available in two different types – Convection (low cost and slow) or Rapid Heat Transfer (high cost and fast).

Convection: 320°F for 1-2 hours depending on load size. **Rapid Heat Transfer:** 375°F for 6 or 12 minutes, depending on the cycle selection.



1



Nylon Pouches with Type 1 Process Indicator Labels
For use with **EVERY LOAD.**

2



Dry Heat Chemical Indicator Strips
For use in **EVERY POUCH.**

3



In-Office SporView® Culture Set
Spore test **AT LEAST WEEKLY**

-OR-



Mail-In ConForm® Biological Indicators
Spore test **AT LEAST WEEKLY**

Ordering Information
1-888-276-7783
customer@crossstex.com

Infection Control Helpline
(1-855-878-3745)
TOLL FREE 1-8558-STERILE
infectioncontrol@crossstex.com

CROSSTEX®
A CANTEL MEDICAL COMPANY



STERILIZATION QUICK GUIDE

Sure-Check® Sterilization Pouches

500/box		100/box	
SCS25	3.5" x 9"	SCL10152	10" x 15"
		SCL10172	10.5" x 17"
200/box		SCL12152	12" x 15"
SCXX2	2.25" x 4"	SCL12182	12" x 18"
SCX2	2.75" x 9"		
SCXS2	3.5" x 5.25"		
SCS2	3.5" x 9"		
SCL3222	3.5" x 22"		
SCW2	5.25" x 6.5"		
SCM2	5.25" x 10"		
SCL5152	5.25" x 15"		
SCL2	7.50" x 13"		
SCL8162	8" x 16"		



Disposable AirView™ II Bowie-Dick Test Pack

30/case
MBD030



Sure-Check® Multi-Parameter Strips

100/bag
SCK | 3.75" x .625"



STEAMPlus™ Type 5* Integrators

100/bag
SSI-100 | 4" Strips



Dry Heat Nylon Self-Seal Pouches

100/box
NSP-400 | 2" x 10"
NSP-405 | 3" x 10"
NSP-410 | 4" x 10"
NSP-420 | 7" x 10.5"
NSP-430 | 9.5" x 13"

Type 1 Process Indicator Labels Included



ConForm® 10 In-Office Biological Monitoring System

C10SK | Digital Incubator and Record Keeper (Specify 110 or 120 Voltage)
C10B125 | Steam Biological Indicators (25/box)
NDB-601 | ConForm® 10 Standard Incubator and Record Keeper



ConForm® 10 Biological Indicators with digital incubator

Dry Heat Chemical Indicator Strip

100/box
DIS-100 | 4" Strips



Mail-In ConForm® Biological Indicators

Premium Test Service - Postage Included

CST060 | 6 Tests (3 Strip Test)
CST120 | 12 Tests (3 Strip Test)
CST480 | 48 Tests (3 Strip Test)

Value Test Service Tests
Postage Not Included

CVT120 | 12 (2 Strip Test)
CVT520 | 52 Tests (2 Strip Test)



SporView® Culture Set

SK-037 | SporView® Culture Set Starter Kit (Dry Heat & EO Gas)*
SK-056 | SporView® Culture Set Starter Kit (Chemical-Vapor)*
CS-020 | SporView® Culture Set (20 tubes and 20 spore strips)
CS-100 | SporView® Culture Set (100 tubes and 100 spore strips)



* Starter kits include box of 20 tubes/strips, dry block incubator (NDB601), record notebook and transfer rack to provide offices with all the necessary components for in-office testing.

*According to ANSI/AAMI/ISO 11140-1:2014

All copyrights and trademarks are property of Crosstex International, Inc.