



This chart is a reference tool for the STERRAD<sup>®</sup> 100NX System. Please refer to the complete User's Guide for additional details.

For more information, contact ASP Customer Support at 1-888-STERRAD or visit [www.us.aspjj.com](http://www.us.aspjj.com).

STERRAD<sup>®</sup> is a registered trademark of Advanced Sterilization Products, division of Ethicon US, LLC, a Johnson & Johnson company.

**ADVANCED STERILIZATION PRODUCTS<sup>®</sup>**

Division of Ethicon, Inc.

a **Johnson & Johnson** company

# How to Properly Dispose of STERRAD<sup>®</sup> 100NX Cassettes

The STERRAD<sup>®</sup> 100NX System's cassette disposal box (20227) can hold up to **2** cassettes before it must be replaced. Please follow the instructions below to safely dispose of your cassettes.

*As a precaution, when handling any part of the system or load items that have been exposed to hydrogen peroxide, please wear the appropriate PPE (chemical resistant latex, PVC/vinyl, or nitrile gloves). Refer to the glove manufacturer's instructions for use for more information.*

***For safety reasons, you must use the cassette disposal box to dispose of cassettes. Never reuse a cassette disposal box.***

## 1. Remove

Open the cassette access door.

Pull the tab on the cassette disposal box to more easily slide it completely out.



## 2. Close & Dispose

Close the lid by pinching it shut along the edge.

Dispose of the closed cassette box according to your facility's policy.

***Never reuse a cassette disposal box.***



## 3. Replace

Insert a new box making sure the lid is open and the tab is facing toward you.

Close the access panel.



To order STERRAD<sup>®</sup> 100NX Cassette Disposal Boxes (20227), please contact an ASP Representative or your local distributor.